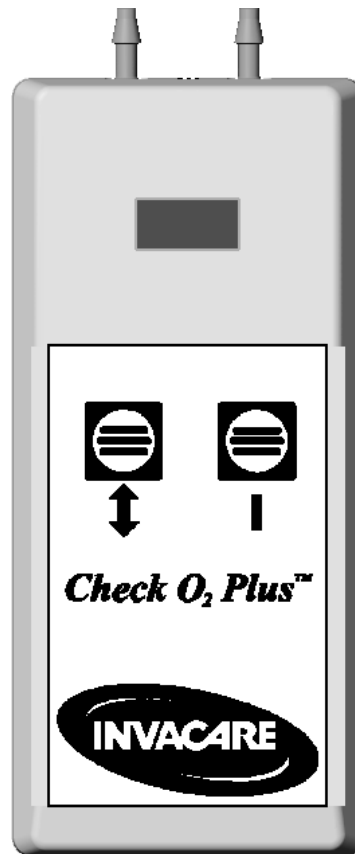
**INVACARE CHECK O<sub>2</sub> PLUS HAND-HELD ANALYZER**  
**ANALIZADOR DE OXIGENO PORTALBE INVACARE CHECK O<sub>2</sub>**  
**ANALYSEUR D' O<sub>2</sub> PORTABLE INVACARE**

**Assembly, Installation and Operating Instructions - ENGLISH**

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**USER: BEFORE USING THIS ANALYZER, READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE.**

---

NOTE: Check all parts for shipping damage. If damage is found, DO NOT use. Contact Carrier/Invacare for further instructions.

## SAFETY SUMMARY

The following recommendations are made for the safe and proper use of the Invacare Check O<sub>2</sub> Plus Hand-Held Analyzer:

### WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

Oxygen rapidly accelerates combustion:

DO NOT smoke while using oxygen.

DO NOT use oxygen in the presence of flames, sparks, or glowing objects.

Ensure that all items are properly and securely installed.

### CAUTION

"Caution: Federal law restricts this device to sale by or on order of a physician, or any other practitioner licensed by the law of the State in which he practices to use or order the use of this device."

DO NOT expose any port of the analyzer to any pressure greater than 20 PSI or damage may result to internal components.

DO NOT use the analyzer on pure oxygen from high pressure cylinders or liquid systems.

DO NOT use the Invacare Check O<sub>2</sub> Plus Hand-Held Analyzer on any gas stream that is humidified. Use of the analyzer with water vapor can cause erroneous readings and internal damage.

## INTENDED USE

NOTE: The Invacare Check O<sub>2</sub> Plus Hand-Held Analyzer is designed and calibrated for use in measuring unhumidified output gas of an oxygen concentrator ONLY.

NOTE: The analyzer is not for continuous monitoring of patient gases.

NOTE: There are no user serviceable parts inside the analyzer. Field calibration of the unit is not required.

NOTE: When using the analyzer after storage in either extremely hot or cold conditions, allow the gas being measured to flow through the analyzer for one (1) minute to bring the internal sensor up to the gas stream temperature.

## INSTRUCTIONS FOR USE (FIGURE 1)

1. Press the ON ( I ) button. The LCD display should come on.

NOTE: The analyzer automatically shuts off after approximately two (2) minutes of operation. This helps save battery life.

2. The display will show "FLO" for approximately two (2) seconds and then switch to readings in L/min.
3. If no reading is displayed or if display is erratic or frozen, refer to the troubleshooting section of this instruction sheet.
4. In order to measure **oxygen concentration** or **flow rate**, connect the analyzer's FLOW/CONCENTRATION inlet with a piece of ¼-inch (I.D.) crushproof tubing to the outlet fitting of the gas source to be measured.
5. In order to measure **pressure**, connect the analyzer's PRESSURE inlet with a piece of ¼-inch (I.D.) crushproof tubing to the outlet fitting of the gas source to be measured.
6. Pressing the FUNCTION (↕) button will toggle the analyzer through its various detecting functions.

| DISPLAY        | MEASURES             | VALUE                  |
|----------------|----------------------|------------------------|
| FLO            | Flow rate            | Liters per minute      |
| O <sub>2</sub> | Oxygen concentration | Percentage oxygen      |
| PSI            | Pressure             | Pounds per square inch |
| HPA            | Pressure             | KiloPascals            |

NOTE: The Invacare Check O<sub>2</sub> Plus Hand-Held Analyzer can be used to measure the performance of an oxygen concentrator from its outlet fitting, or any concentrator accessory such as the Invacare Remote Flowmeter or Pediatric Flowmeter. However, Invacare does NOT recommend using the Invacare Check O<sub>2</sub> Plus Hand-Held Analyzer to measure flow rates below 1 L/min. See product specifications.

NOTE: The concentrator output gas, measured from the concentrator outlet fitting or from a concentrator accessory outlet fitting, must NOT contain humidified gas. Disconnect any humidifier bottle before taking any readings to prevent erroneous readings and damage to the internal flow sensor. Be sure to reconnect any required humidifier bottle before returning any equipment to patient use.

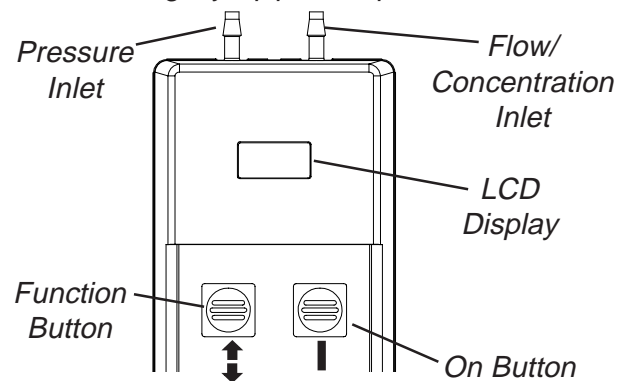
FIGURE 1-INSTRUCTIONS FOR USE

7. With "O2" readings, values below 70% will be displayed as "LO".
8. When the battery is completely discharged, **BAt** will flash once on the LCD and the analyzer will turn itself off. Refer to section on changing the battery in this Instruction sheet.

## CHANGING THE BATTERY

*NOTE: The internal battery needs to be replaced whenever the **BAt** display is seen on the LCD.*

1. Turn the analyzer over.
2. Remove the six (6) screws holding the case bottom to the top and take the bottom cover off.
3. Remove the discharged battery from the battery clips and dispose of or recycle properly.

**CAUTION**  
Observe proper polarity for battery installation.

4. Install a new 9 volt battery on the battery clips. Make sure the battery snaps tightly on the clips.

*NOTE: Use of a DURACELL® alkaline 9 volt battery will provide for maximum life of the unit.*

5. Turn the unit over and press the ON button. The display should come on. If it does, replace the bottom cover and install the six cover screws.
6. If the unit does not come on, remove the battery, check it in another device or on a battery tester and reinstall it. Try turning the unit on again.
7. If the unit still fails to operate with a known good battery, contact Invacare Technical Service for repair.

## TROUBLESHOOTING

*NOTE: There are no user serviceable parts inside the analyzer.*

1. If the unit fails to come on, replace the battery with a known good battery.
2. If the unit fails to come on with known good battery, contact Invacare Technical Service for repair.
3. If the unit comes on but the FUNCTION button does not select any functions, contact Invacare Technical Service for repair.
4. If the display comes on but the readings are erratic or frozen, contact Invacare Technical Service for repair.
5. If a segment of the display is missing or displayed intermittently, contact Invacare Technical Service for repair.

## PRODUCT SPECIFICATIONS

|                                     |  |
|-------------------------------------|--|
| <b>OPERATIONAL</b>                  |  |
| <b>Pressure:</b>                    |  |
| Range:                              | 0 to 10PSIG    0 to 68.9 kPa                           |
| Resolution:                         | 0.1 PSI        0.7 kPa                                 |
| Accuracy:                           | ± 2% F.S.     ± 2% F.S.                                |
| <b>Flow:</b>                        |  |
| Range:                              | 0 to 6.0 L/min.  |
| Input:                              | Unhumidified oxygen concentrator output only           |
| Resolution:                         | 0.1 L/min.   |
| Accuracy:                           | ± 5% F.S. (± 0.3 L/min.)                               |
| <b>Gas Stream</b>                   |  |
| Temperature:                        | +59 to +95°F / +15 to +35°C                            |
| <b>Concentration:</b>               |  |
| Range:                              | 73% to 96% O <sub>2</sub>                              |
| Input:                              | Unhumidified oxygen concentrator output only           |
| Resolution:                         | 0.1 %  |
| Accuracy:                           | ± 2%   |
| <b>Gas Stream</b>                   |  |
| Temperature:                        | +59 to +95°F / +15 to +35°C                            |
| <b>Alarm:</b>                       | Low battery  |
| <b>Power On Time:</b>               | 1 minute, 45 seconds ±5 seconds                        |
| <b>Display:</b>                     | LCD  |
| <b>Battery:</b>                     | Single NEDA/ANSI 1604A alkaline 9 volt                 |
| <b>Battery life:</b>                | 400+ hours (13,700+ read cycles) with alkaline battery |
| <b>Warranty:</b>                    | 2 years  |
| <b>PHYSICAL:</b>                    |  |
| Weight:                             | 10 oz. (295g)  |
| Width:                              | 3.3 in. (8.38 cm)                                      |
| Length:                             | 7.3 in. (18.54 cm)                                     |
| Length Including Inlet Fittings:    | 7.9 in. (20.07 cm)                                     |
| Thickness:                          | 1.25 in. (3.18 cm)                                     |
| <b>ENVIRONMENTAL</b>                |  |
| <b>Operating Temperature Range:</b> | +59 to +95°F / +15 to +35°C                            |
| <b>Storage Temperature Range:</b>   | -4 to +158°F / -20 to +70°C                            |
| <b>Humidity range:</b>              | 20 to 60% relative humidity                            |
| <b>Altitude (maximum):</b>          | Up to 6000 ft. (1828 meters) above sea level           |

---

## **LIMITED WARRANTY**

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its parts to be free from defects in materials and workmanship for a period of two (2) years from date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact your supplier or dealer.

DO NOT return products or parts directly to our factory without prior written consent. Any such shipments will be refused.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE CORPORATION'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE CORPORATION OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE CORPORATION'S CONTROL. SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE FOLLOWING INSTRUCTIONS.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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